



## OVERVIEW OF ALL FUNCTIONS AND ADVANTAGES

- design suitable for children
- visual feedback of the folding balloon motivates the children and helps the parents
- inhalation flow and volume control optimise the inhalation manoeuvre
- great time saving of only one necessary inhalation
- easy handling and cleaning


Using the Watchhaler makes inhalation therapy effective and convenient!

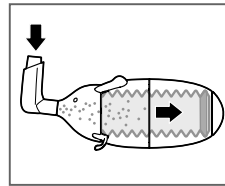

1. Place the metered dose inhaler (MDI) onto the Watchhaler mouthpiece and release one puff

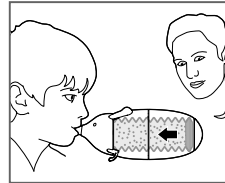

2. Breathe out completely and put the mouthpiece into the mouth

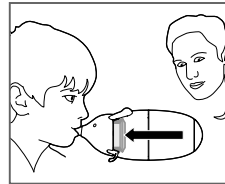

3. Let the child suck one time at the mouthpiece until the folding balloon is compressed completely

## URNS INHALATION INTO A CHILD'S PLAY

Would you like to know more about the Watchhaler?  
Please visit our web site or contact us by phone.

- Internet: [www.watchhaler.com](http://www.watchhaler.com)

www.agenturzz.de

# URNS INHALATION INTO CHILD'S PLAY

Watchhaler – the innovative holding chamber for your child with visual feedback and easy handling



**ACTIVAERO**  
TECHNOLOGIES

Activaero GmbH  
Wohraer Str. 37  
35285 Gemünden - Germany  
Phone: +49 6453 | 64818 0  
Fax: +49 6453 | 64818 22  
email: [service@watchhaler.de](mailto:service@watchhaler.de)

CE

  
**WATCHHALER™**

## WATCHHALER – DESIGN AND FUNCTION INNOVATION


With the Watchhaler a new time period in the inhalation therapy is started.

While current spacers are designed in a sterile medical approach, the Watchhaler is designed as a coloured toy. This makes the inhalation treatment for the children convenient. Plus the Watchhaler solves existing coordination problems, reduces electro-static charge and gives a visible feedback. This makes it very unique and innovative!

The Watchhaler has a transparent body with a foldable green balloon inside. The balloon is used as a temporary storage for the drug aerosol and is folded together during an inhalation. A successful inhalation is visible for both the children and parents.

A valve controls the inspiration speed during the inhalation and helps to achieve a slow, more effective and convenient inhalation.

## IT IS CHILD'S PLAY EASY HANDLING AND CLEANING

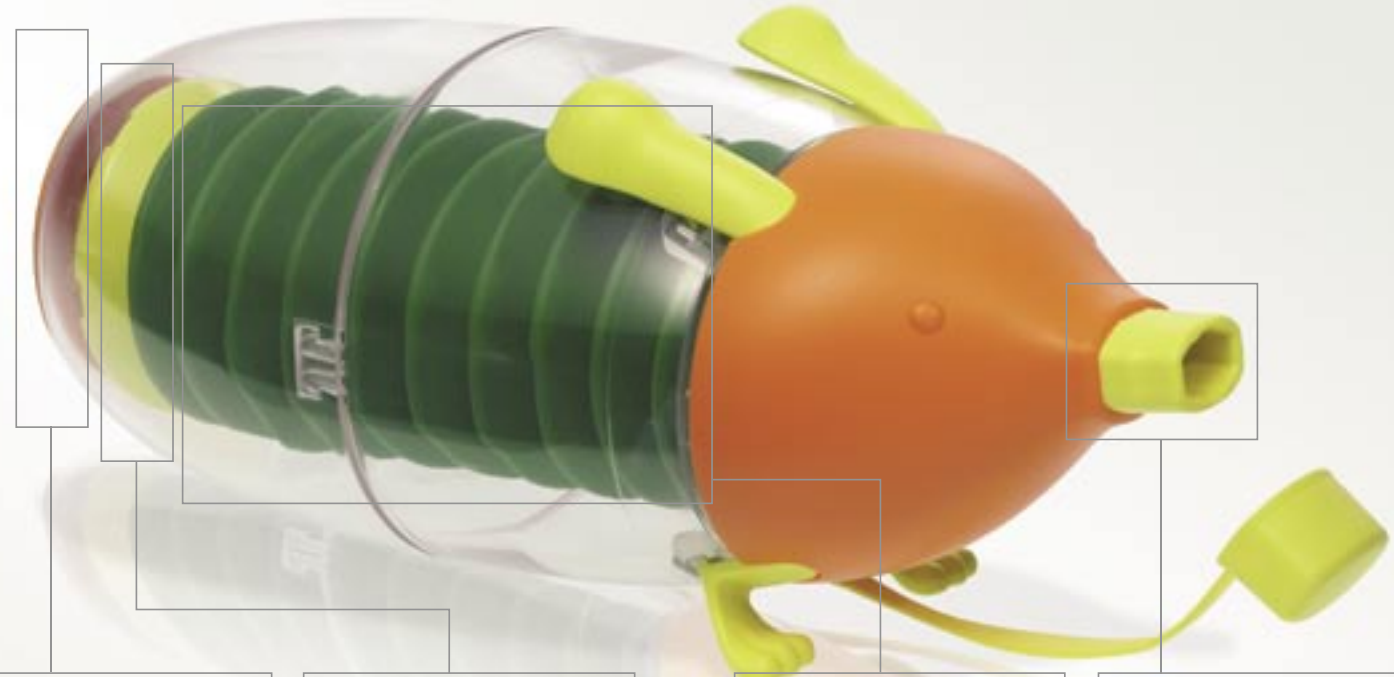

### FLOW VALVE

The limited valve optimises the inhalation flow. This helps the child to inhale slowly and consistently which allows a constant and reliable delivered drug amount into the lung.

### RING

The yellow ring on the balloon is a visual feedback and motivates the child to inhale in an optimal way and also helps the parents to identify a successful inhalation.

### FOLDING BALLOON

The transparent device body and the folding balloon are designed for children for better patient compliance. This lay-out also reduces the problem with electro-static charge. The balloon standardises the inhalation volume and can be emptied with only one breath.

### MOUTHPIECE

The mouthpiece fits to most common used metered dose inhalers (MDIs). The size is perfect for small children mouths.



# WATCHHALER™

[WWW.WATCHHALER.COM](http://WWW.WATCHHALER.COM)

